



INSTRUCTIONS FOR USE - MPC head lice comb

For detecting and removal of head lice, eggs and nits.

1. Brush any snarls or tangles out of the hair prior to combing
2. Comb one section of hair (comb wide) at a time, from root to tip
3. Wipe off any lice, eggs or nits on the comb with a tissue
4. Continue to comb until no more lice can be found
5. After use, wash the comb with lukewarm water and hand soap

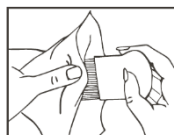

 Do not use if damaged or miscoloured. Do not boil.

This product is a medical device and is reusable.

If any serious incident has occurred in relation to the use of the comb, you should report it to the distributor, the manufacturer and/or the competent local medical device authority.

Made in EU.

Material: Akulon PA6 and stainless steel.



 Zantey ApS • Lejrvej 17 • 3500 Værløse • Denmark • www.zantey.com

Zantey ApS_IFU MPC_Rev. 20200604_004